

AT Tech Tip: Recording Messages on the iTalk2 Communicator



InterACT Team

The Interdisciplinary Augmentative Communication & Technology Team



iTalk2 Communicator

1. Turn device on using the volume control knob on the back of the device.
2. Press the record button (located on the back of the device) and a desired colored switch surface simultaneously.
3. Speak your message while **continuing to hold** the record button and the desired colored switch surface.
4. Release both the record button and the colored switch surface when you're done speaking your message.
5. Follow steps 2-5 to record a message for the second colored switch surface.

***Requires one 9-volt battery.**