

AT Tech Tip: Recording Messages on the TechTalk and the Tech II 8



InterACT Team

The Interdisciplinary Augmentative Communication & Technology Team



TechTalk and the Tech II 8 (8-location device)

1. Turn the device on using the volume control knob on the back of the device.
2. Use the level select knob to select the level you wish to record onto.
3. Once on the desired level to program, press and hold the red “rec” button (located on the back of the device) and a cell on the device simultaneously.
4. Speak your message **while continuing to hold** both the record button and the desired cell.
5. Release both the record button and the cell when you’re done recording your message for the selected cell.
6. Follow steps 3-5 to record messages on all cells for the desired level.
7. Follow steps 2-5 to record messages for another level.

***Requires 4 AA batteries.**